OPED GmbH

Medizinpark 1
83626 Valley/Oberlindern
Germany
Fon +49 (0)80 24/60 81 8-210
Fax +49 (0)80 24/60 81 8-299
mail@oped.de
www.oped.de

OPED AG

Hinterbergstrasse 26
6330 Cham
Switzerland
Fon +41 (0)41/748 53 35
Fax +41 (0)41/748 53 40
contact@oped.ch
www.oped.ch

OPED UK Ltd

Unit 2 Glenmore Business Centre
Waller Road
Hopton Industrial Estate
Devizes
SN10 2EQ
United Kingdom
Phone +44 (0)1380 722177
Fax +44 (0)1380 710095
enquiries@oped.biz
www.opeduk.co.uk

OPED Australia Pty Ltd

2/97 Montague St.
North Wollongong NSW 2500
Australia
Phone +61 (0)2 4226 5880
Fax +61 (0)2 4226 5881
contact@oped-australia.com.au
www.oped.com.au

OPED Asia-Pacific Co., Ltd.

5th Floor, No 16, Lane 148
Lide Street, Zhonghe Dist.
New Taipei City 23512
Taiwan
Phone +886 2 222 65586
Fax +886 2 222 65520
contact@oped-asia.com
www.oped-asia.com

WM-TEVP-03-6 | 03/2015 | attraction | Brunner Druck & Design

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
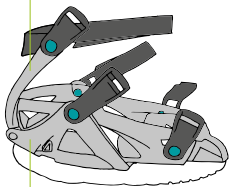
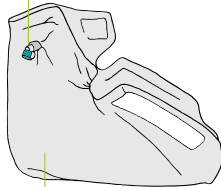
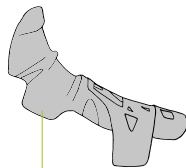
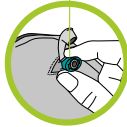
E First, pull up the valve ring. Then, place the foot in the fabric cover and close it.

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
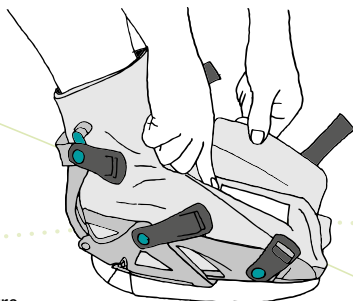
E Attach the upper shell and put on the heel bracket. Close the straps, starting with the one in the middle.

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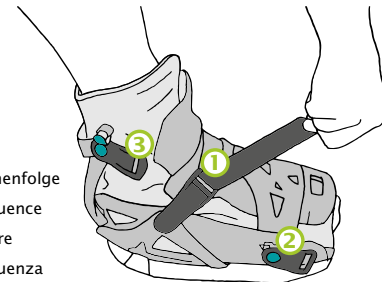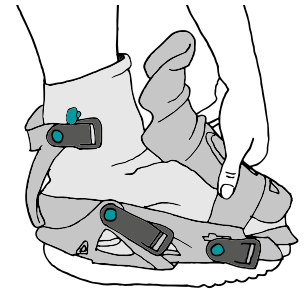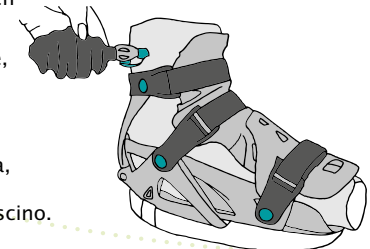
E Attach the hand pump to the valve, and pump until all the air has been removed from the cushion.

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
E Pull the blue valve ring outwards and maneuver it left and right until the cushion becomes soft.

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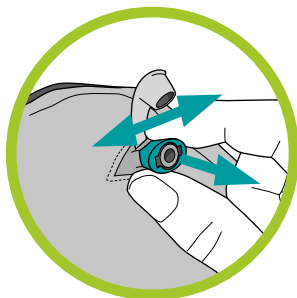
E Press the blue valve ring down, attach the bulb pump the the exposed tip and pump until the bulb no longer inflates.

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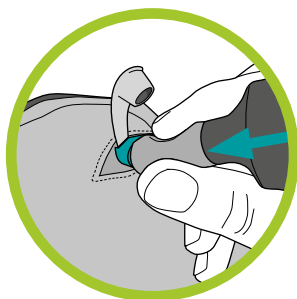
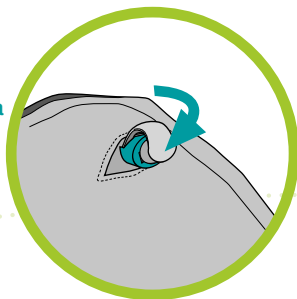
E Open the back zip completely. Remove the vacuum cushion. Make cushion soft by opening valve. Smooth out the vacuum-cushion with your hand. Pump out air to make vacuum cushion-hard. Put the vacuum cushion back into the new cover and insert the valve through the buttonhole

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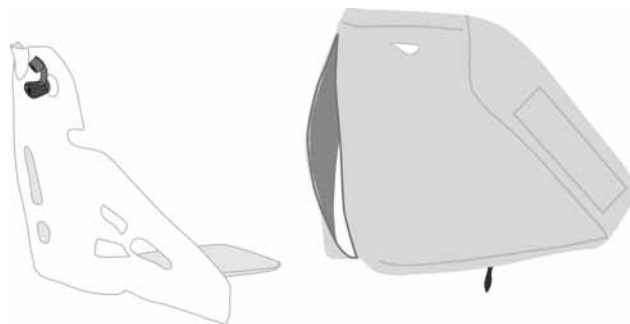E Press both release buttons simultaneously.

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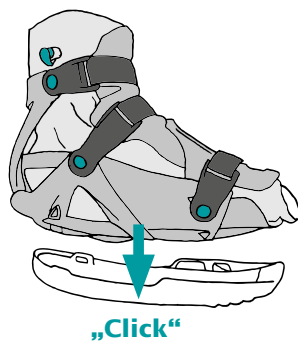
E Place the VACOpedes directly into the center of the sole, pressing hard against it until two clicks are heard.

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
E Remove the sole (see removing the sole). Press the release button. Pull the toe piece to the required length.

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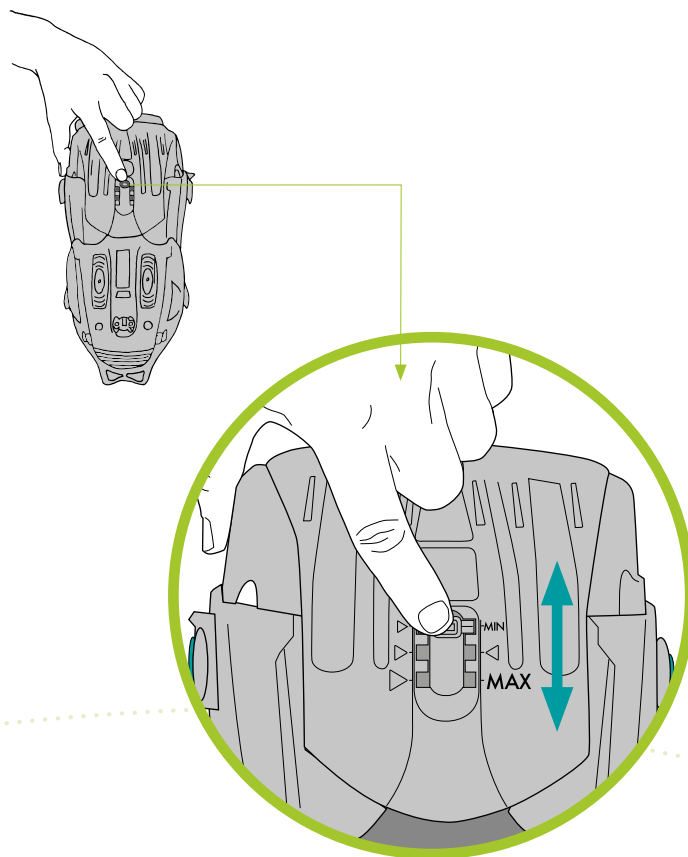Useful Details

Washing Directions

- Shell: Dishwasher at 60° C
- Straps: Machine wash at 40° C
- Terry cloth covers: Machine wash at 60° C (ensure zipper is closed)
- Vacuum cushion: Disinfectant spray or wipes
- Storage: Dry condition

Product features

- Can be worn on the right or left foot.
- Sizes:

	Small	Medium	Large
EUR	35–39	39–44	44–50
UK	2 ½–6	5 ½–10	9 ½–14
US (M)	3 ½–7	6 ½–10 ½	10 ½–15
US (W)	4 ½–7 ½	7 ½–11 ½	11 ½–16

Indications / applications

- Metatarsal fractures I - IV
- Hallux valgus/rigidus
- Arthrodesis of toe joint
- Hammer and claw toes
- Fractures of the big toe

Troubleshooting/ Safety notes

Pressure points

If at any point you feel any discomfort while using the VACOpedes, try the following:

- Remove the VACOpedes completely, and re-aerate the vacuum cushion. Redistribute the styrofoam beads by shaking the cushion and smoothing it with your hands, before reapplying the boot.
- Re-aerate the vacuum cushion, and inspect the specific area causing discomfort, using your fingers to ensure styrofoam beads are properly separating the tender spot from the outer shell.

If neither of these adjustments assist in relieving pain, or if numbness in the toes or feet occur and /or your skin turns bluish, it is important you contact your doctor immediately. For more useful tips visit foot.oped.de

The VACOpedes is designed specifically for foot injuries. Do not jump or attempt fast walking or running while wearing the VACOpedes. If you should fall while wearing the VACOpedes, check it for any damage.

Technical Data/ Packing dimensions

Materials used

Hard shells (PP), covers (cotton/PU foam), cushion cover (PVC), filling material (EPS beads) straps (PP fabric), outer sole (EVA), upper shell (EPS), clasp of upper shell (POM)

Weight

- 950 g

Dimensions

- 28 x 33 x 18 cm

Warranty

The product warranties comply with all legal regulations.

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